STERILEEO

en Do not re-use	en Use-by (Expiry date) YYYY-MM-DD
cs Pro jednorázové použití	cs Použijte do (datum použitelnosti) RRRR-MM-DD
da Må ikke genbruges	da Anvendes inden (udløbsdato) ÅÅÅÅ-MM-DD
de Nicht wiederverwenden	de Verwendbar bis (Verfallsdatum) JJJJ-MM-TT
es No reutilizar	es Fecha de caducidad AAAA-MM-DD
fi Älä käyttää uudelleen	fi Viimeinen käyttöpäivä VVVV-KK-PP
fr Ne pas réutiliser	fr Utiliser jusqu'au (date de péremption) AAAA-MM-JJ
hr Nije za višekratnu uporabu	hr Upotrijebiti do (rok valjanost) GGGG-MM-DD
hu Újrafelhasználni tilos	hu Felhasználható (lejárati idő) ÉÉÉÉ-HH-NN
it Non riutilizzare	it Utilizzare entro (data di scadenza) AAAA-MM-GG
nl Niet opnieuw gebruiken	nl Te gebruiken tot (vervaldatum) JJJJ-MM-DD
no Må ikke brukes om igjen	no Bruk innen (utløpsdato) ÅÅÅÅ-MM-DD
pl Nie używać ponownie	pl Należy wykorzystać do (data ważności) RRRR-MM-DD
pt Não reaproveitar	pt Utilizar até (Prazo de validade) AAAA-MM-DD
sk Nepoužívajte opakovane	sk Dátum spotreby (dátum expirácie) RRRR-MM-DD
sl Samo za enkratno uporabo	sl Uporabiti do (rok uporabe) LLLL-MM-DD
sv Får ej återanvändas	sv Utgångsdatum (ÅÅÅÅ-MM-DD)
zh 不可重新使用	zh 使用截止日期 (过期日期) 年-月-日
ar لا يُعاد استعماله	ar يُستخدم قبل (تاريخ انتهاء الصلاحية) يوم/شهر/سنة
he לא לשימוש חוזר	he לשימוש עד (תאריך תפוגה) YYYY-MM-DD

en Sterilized using ethylene oxide	en Single sterile barrier system
cs Sterilizováno pomocí ethylenoxidu	cs Systém jednovrstvé sterilní bariéry
da Steriliseret med ethylenoxid	da Enkelt sterilt barrieresystem
de Mit Ethylenoxid sterilisiert	de Einfaches Sterilbarrieresystem
es Esterilizado con óxido de etileno	es Sistema de barrera estéril simple
fi Steriloitu etyleenioksidilla	fi Yksittäinen steriili estejärjestelmä
fr Stérilisé à l'oxyde d'éthylène	fr Système de barrière stérile simple
hr Sterilizirano etilen oksidom	hr Sustav s jednostrukom sterilnom barijerom
hu Etlénoxiddel sterilizálva	hu Egyszeres steril gátrendszer
it Sterilizzato con ossido di etilene	it Sistema di barriera sterile singolo
nl Gesteriliseerd met ethylenoxide	nl Enkel steriel barrieresysteem
no Steriliseret med etylenoksid	no System med én steril barriere
pl Sterylizowany przy użyciu tlenku etylenu	pl System pojedynczej bariery sterylnej
pt Esterilizado com óxido de etileno	pt Esterilizado com óxido de etileno
sk Sterilizované etylenoxidom	sk Jednoduchý sterilný bariérový systém
sl Sterilizirano z etilen oksidom	sl Enojni sterilni pregradni sistem
sv Steriliserad med etylenoxid	sv Enkelt sterilt barrierisystem
zh 已使用环氧乙烷灭菌	zh 单次无菌屏障系统
ar مُعَمَّم باستعمال أكسيد الإيثيلين	ar نظام حاجز معقم واحد
he עוקר באמצעות אתילן אוקסיד	he מערכת מחסום עוקר יחיד

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www.infusion-set.com

Ypsomed AG
Brunnmattstrasse 6
CH-3400 Burgdorf
Switzerland

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English

Indications for use:

The YpsoPump® Inset infusion set is indicated for the subcutaneous infusion of insulin administered by the external infusion pump YpsoPump®. The infusion set is indicated for single use.

Intended Use

The infusion set is indicated for subcutaneous infusion of insulin in the treatment of diabetes. The infusion set has been tested to be compatible with insulin approved for subcutaneous infusion.

Description

YpsoPump® Inset is an infusion device which combines an infusion set with an aid for infusion (serter). As the infusion set is an integrated part of the serter, YpsoPump® Inset is delivered assembled and ready for use as a single, sterile unit.

Contraindications

YpsoPump® Inset is neither intended nor indicated for use in intravenous infusion of blood or blood products.

WARNINGS

- Carefully examine the instructions for use before inserting YpsoPump® Inset. Failure to follow instructions may result in pain or injury.
- YpsoPump® Inset is only sterile and non-pyrogenic if the insertion device is undamaged and unopened. Do not use if the insertion device is already open or has been damaged. Ensure sterility by checking if the sterile paper and tamper-proof seal are undamaged.
- When using YpsoPump® Inset for the first time, do so in the presence of a healthcare provider. Since different individuals have varying amounts of subcutaneous layer, ensure that an appropriate cannula length is chosen to fit your needs.
- Inaccurate medication delivery, site irritation or infection may result from improper insertion or maintenance of your infusion site.
- Do not re-insert the introducer needle into the infusion set. This could cause a tearing or damage of the soft cannula and may lead to unpredictable medication flow.
- Reuse of the infusion set may cause site irritation or infection and may lead to inaccurate medication delivery.

- Be sure that the needle guard is removed before insertion is made.
- If the soft cannula bends during insertion, apply a new YpsoPump® Inset immediately at a new site.

PRECAUTIONS

- Never try to fill or free a clogged tubing while the infusion set is inserted. This may result in unpredictable medication flow.
- When unwinding, release the tubing with caution as a hard pull of the tubing can result in damage to the infusion device. Ensure that the infusion set is properly in place in the serter when the tubing is fully released – see picture 8.
- Do not leave air in the infusion set. Make sure to fill the tubing completely. Consult pump manufacturer's instructions for use.
- YpsoPump® Inset is a single use device and should be disposed of immediately after use. Do not clean or re-sterilize.

- Do not in any way put disinfectants, perfumes, deodorants, cosmetics or other products containing alcohol or disinfectants in contact with the connector or the tubing. This may impact the integrity of the infusion set.
- Replace the infusion set if the adhesive tape becomes loose or is displaced from its original site. Since the cannula is soft, it will not cause any pain if it becomes dislodged, and this may take place without notice. The soft cannula must always be completely inserted to receive the full amount of medication.
- Replace the infusion set every two to three days, or per your healthcare provider's instructions.
- If your insertion site becomes irritated or infected, replace the infusion set and use a new site until the first site has healed.
- Wash your hands before disconnecting and reconnecting the infusion set. Consult your healthcare provider on how to compensate for missed medication when disconnected.
- Never point a loaded insertion device towards any body part, where insertion is not desired.

RECOMMENDATIONS

- Check your blood glucose level one to three hours after insertion of the YpsoPump® Inset. Measure your blood glucose on a regular basis. Discuss this with your healthcare provider.
- Do not change your infusion set just prior to bedtime, unless blood glucose can be checked one to three hours after insertion.
- If your blood glucose level becomes unexplainably high or an occlusion alarm occurs, check for clogs and leaks. If in doubt, change your infusion set since the soft cannula could be dislodged or partially clogged. Discuss a plan to bring down your blood glucose level with your healthcare provider should any of these problems arise. Test your blood glucose level to make sure the problem is corrected.
- Carefully monitor your blood glucose level when disconnected and after reconnecting.
- If, during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

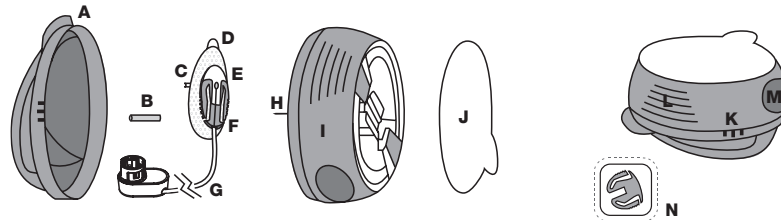
Storage and Disposal

- Store infusion sets in a cool, dry location at room temperature. Do not store infusion sets in direct sunlight, or in high humidity.
- Store and handle insulin according to the manufacturer's instructions.
- Put the lid back on the insertion device and dispose in a proper sharps container according to local laws, to avoid risk of needle stick.
- Dispose of a used infusion set according to local regulations for biohazardous waste.

English

Preparation and Insertion

Read all instructions carefully before using the YpsoPump® Inset infusion set. For information about pump therapy, possible errors, and potential risks and their causes, consult your pump manufacturer's instructions for use before connecting YpsoPump® Inset. When using YpsoPump® Inset follow proper hygiene procedures. Please see the corresponding illustrations on page 88–90.

Contents

- A** Lid
- B** Needle guard
- C** Soft cannula
- D** Adhesive tape
- E** Cannula housing
- [F]** Connector needle
- [G]** Tubing
- [H]** Introducer needle
- [I]** Insertion device
- [J]** Sterile paper

Important note!

- [K]** Three raised dots – press here when lifting the lid off (see Figure 6)
- [L]** Lined indentations – press here when loading YpsoPump® Inset (see Figure 10)
- [M]** Round indentations – press here simultaneously when releasing YpsoPump® Inset (see Figure 12)
- [N]** Disconnect cover (see Figure B)

INSTRUCTIONS

Unfold the cover of this booklet to view images that correspond to the steps below.

- 1** Wash your hands before inserting YpsoPump® Inset.
- 2** Suggested insertion sites. Choose an insertion site as recommended by your healthcare provider. Do not use an area right next to your previous insertion site.
- 3** Clean the infusion site with a disinfectant as directed by your healthcare provider. Be sure the area is air dry before inserting YpsoPump® Inset.
- 4** Pull the red tape to remove the protective seal.
- 5** Remove the sterile paper.
- 6** Press the three raised dots on each side of the lid firmly with one hand and lift up the lid with the other hand.

CAUTION: Be careful not to bend or touch the YpsoPump® Inset introducer needle when preparing the serter and infusion set.

Important note! If the infusion set is not securely placed in the insertion device with the needle pointing straight ahead prior to insertion, pain or minor injury may occur. To avoid this, please ensure that the following steps are taken gently so the infusion set is not loosened/removed by accident.

- 7** **Unwinding the tubing:** Carefully twist the beginning of the tubing out of the slot. Now unwind the tubing from the insertion device by pulling it carefully upwards.

CAUTION: Do not touch the YpsoPump® Inset introducer needle when unwinding the tubing. Be careful not to pull too hard when the tubing is fully released. You could accidentally remove the infusion set from the introducer needle.

- 8** **Before insertion:** Attach the adapter of the infusion set straight onto the cartridge compartment of the YpsoPump®. Push the adapter gently down to the introducer needle. Turn the adapter in a clockwise direction until you feel it slide into lock position. Check visually that the adapter is attached firmly and aligned with the pump case. After connection, make sure air bubbles are removed by priming the tubing assembly according to the priming instructions of the YpsoPump®.

Priming volume for the tubing with adapter (U-100 insulin, volumes are approximate):

46 cm (18 inch):	10 units (0.10 ml)
60 cm (24 inch):	12 units (0.12 ml)
80 cm (31 inch):	14 units (0.14 ml)
110 cm (43 inch):	18 units (0.18 ml)

CAUTION: When priming the infusion set, hold YpsoPump® Inset with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

- 9** Pull up gently to remove the adhesive backing paper.

10 Prepare the inserter: Place fingers on lined indentations. Press lined indentations on each side. Pull spring until you hear a “click”.

- 11** Carefully remove the needle guard by gently twisting and then pulling it. Check that the soft cannula does not exceed the introducer needle.

CAUTION: Before positioning YpsoPump® Inset over the cleaned infusion site, place the tubing in the slot on the side of the inserter to ensure that the tubing is not caught underneath the infusion set during insertion.

12 Insert YpsoPump® Inset: Position YpsoPump® Inset over the insertion site. Simultaneously press the round indentations on both sides of the insertion device to insert YpsoPump® Inset.

- 13** Push gently on the center of the inserter to secure the adhesive on the skin.

- 14** Remove the serter and introducer needle by grasping the center and pulling it gently back. Gently massage the adhesive onto the skin. Fill the empty space in the cannula with the following amount of insulin:

U-100 insulin:

6 mm:	0.1 units (0.001 ml)
9 mm:	0.2 units (0.002 ml)

- 15** Push the lid back in place until you hear a “click.”

Disconnect

A YpsoPump® Inset allows you to temporarily disconnect your pump without having to change the infusion set. Hold the adhesive in place by placing a finger just in front of the cannula housing, and gently squeeze the sides of the connector. Pull the connector needle straight out from the cannula housing.

B Insert the disconnect cover into the cannula housing until you hear a “click”.

Reconnect

C If necessary, prime the infusion set until medication dispenses from the needle.

CAUTION: When priming the infusion set, hold YpsoPump® Inset with the needle pointing down.

D Remove the disconnect cover from the cannula housing by pressing the grips. Place a finger on the adhesive tape just in front of the cannula housing.

E Push the connector needle straight in until you hear a “click”.

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